OCULUS

Universal Hubtische

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→ Protect packaged kits from moisture.

Safety instructions

The information of this assembly instructions are only valid in conjunction with the instruction manual for your OCULUS lifting table.

- Read the assembly instructions carefully. For further assistance with assembly, we recommend our video instructions. QR code with direct link on the back of these instructions.

Observe the following safety instructions:

- Keep the assembly instructions carefully.
- During assembly, ensure that proper grounding is provided between all components.
- Only Germany: Only set up the Ophthalmic Table in medically used rooms if they have been installed in accordance with VDE regulations 0100-710.
- Avoid dripping, splashing and splashing water near the Ophthalmic Table and ensure that no liquid can penetrate the Ophthalmic Table. Therefore, do not place any containers filled with liquid near or on the Ophthalmic Table.
- Do not install the lifting table in areas at risk of explosion, in the presence of flammable anesthetics or volatile solvents such as alcohol, petrol or similar.
- Do not install the lifting table in an operating room.
- Before using the lifting table: Read the instruction manual for the OCULUS lifting table!

Safety symbols

Indicates important instructions

Identifies a potentially dangerous situation which may cause minor injury.

Time Expenditure

Depending on the configuration of the ophthalmic table, the estimated time required to assemble a table varies between 25-45 minutes.


Scope of delivery

- If you notice any transport damage upon delivery, please report this immediately to the transport company.



Standard accessories Ophthalmic Table	Universal Ophthalmic Table	System Ophthalmic Table
2 device connection cable	X	
Adjustable device feet	X	X
Universal cable glands	X	
Cable glands specifically for OCULUS devices		X
Drillings for mounting OCULUS devices		X
Electrical unit with isolation transformer		X


Optional accessories Ophthalmic Table	Universal Ophthalmic Table	System Ophthalmic Table
Monitor mount (optional)		X
Rollers (optional)	X	X

additional installations (optional) as part of the OCULUS device	Universal Ophthalmic Table	System Ophthalmic Table
Device power supply		X
Hard drive		X
Y cable		X
Chin-forehead rest		X
Cograil		X
Cover cograil		X

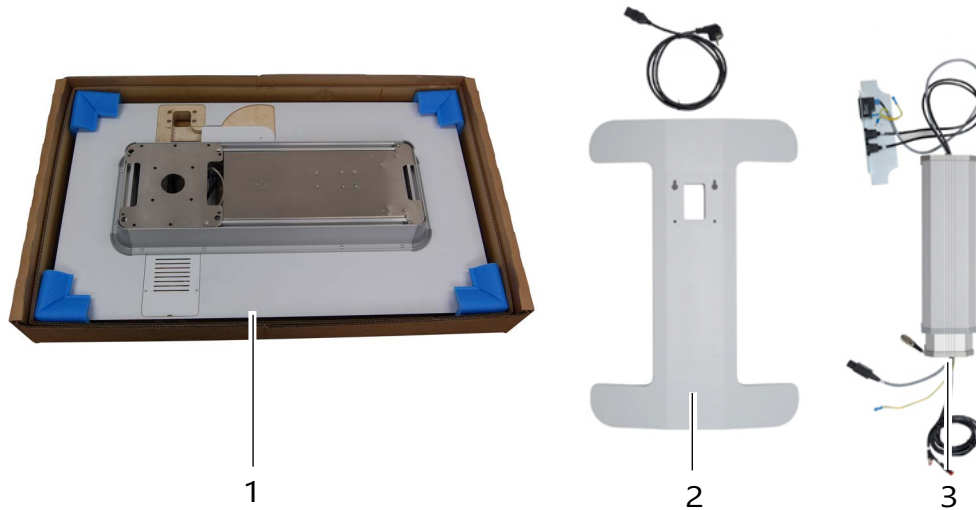
No.	Description
	Kit 1
1	Table top
	Kit 2
2	Table base
3	Lifting column with connector plate
4	Instruction manual
5	Replacement fuses
6	Mounting set
7	Cover plate for cable entry
8	Cover cable tunnel
9	Assembly instruction

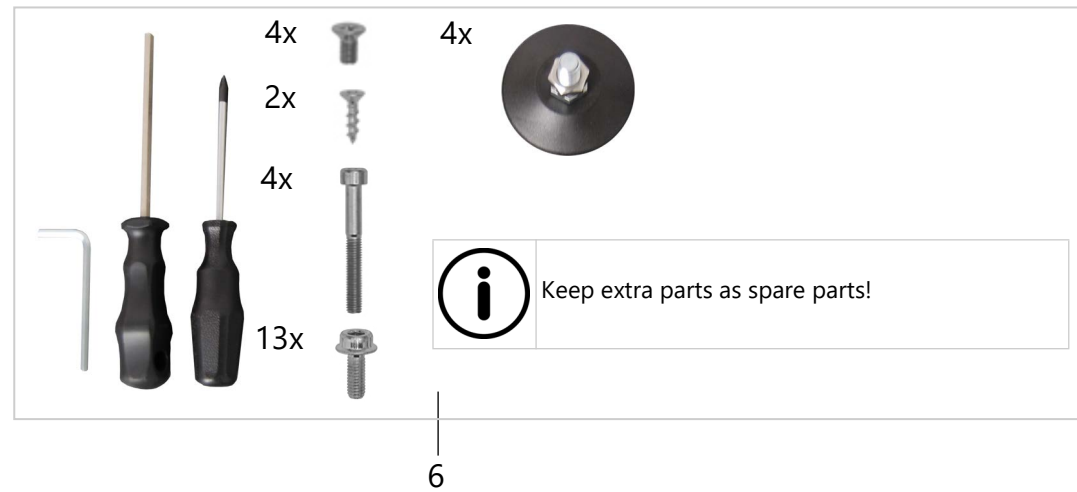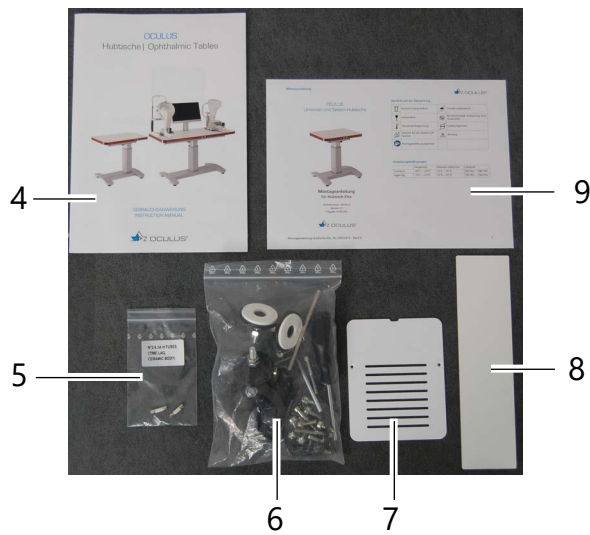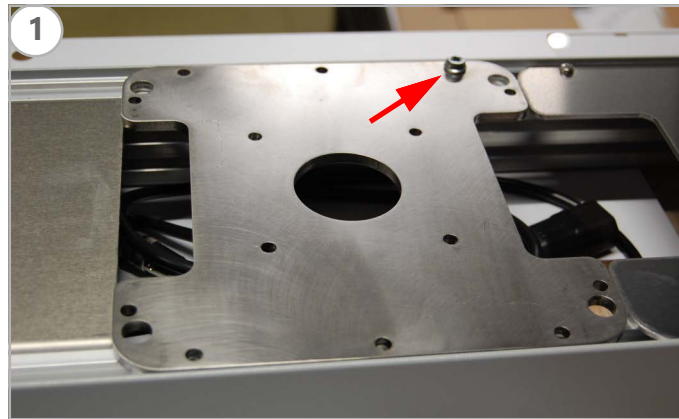- Leave the table top upside down in the box.
- Loosen the locking screw on the bottom side, put it aside and store it for later use.
- Remove the screw plate.

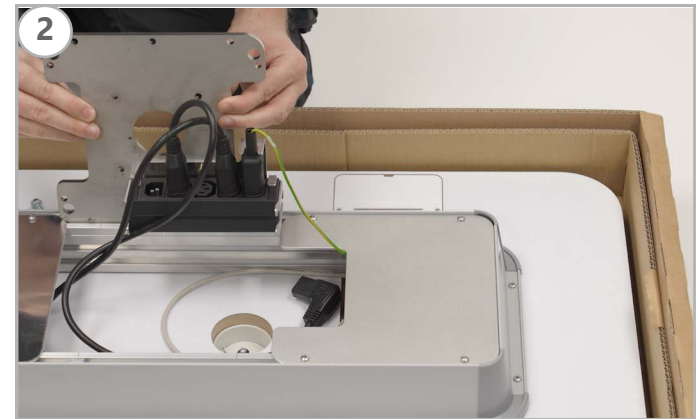
- The retaining clip with plug strip is screwed to the screw plate for transportation.


- Unscrew retaining clip.


- Put retaining clip with plug strip into the frame.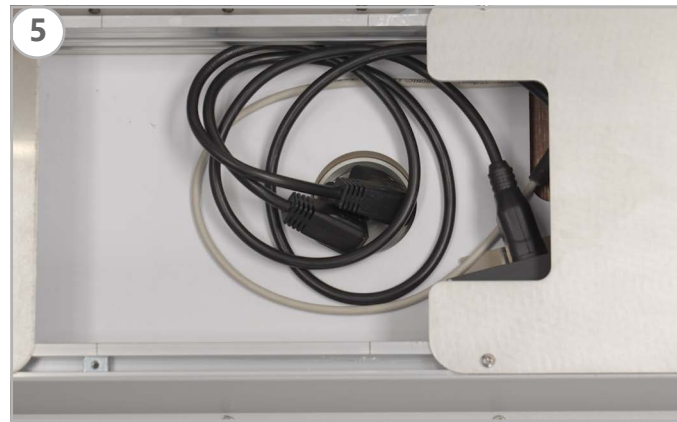

→ Place the plugs in the cable gland.

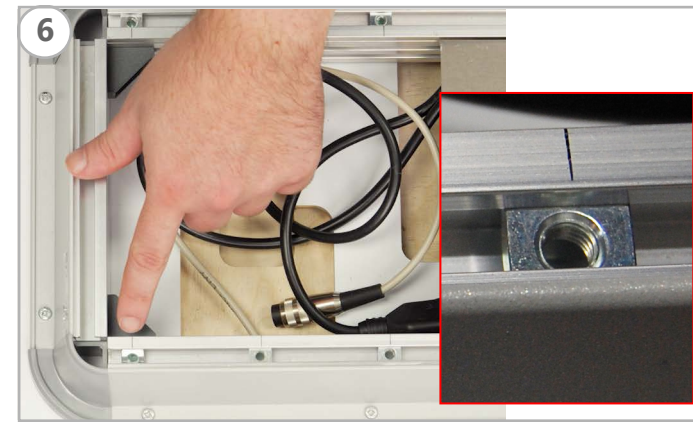→ Make sure that the countersunk nuts are positioned correctly in the slide rail.

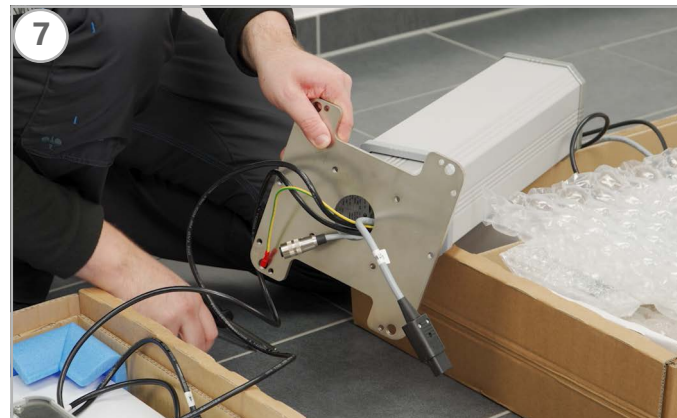
- Remove the lifting column from the box of kit 2 and lay it ready.
- Route all cables through the hole in the screw plate.

- Screw on tightly the screw plate onto the lifting column using 4 x M6x16 Allen screws, use the second tool to apply more force.

→ Place the lifting column on the upturned table top, being careful with the connector plate.

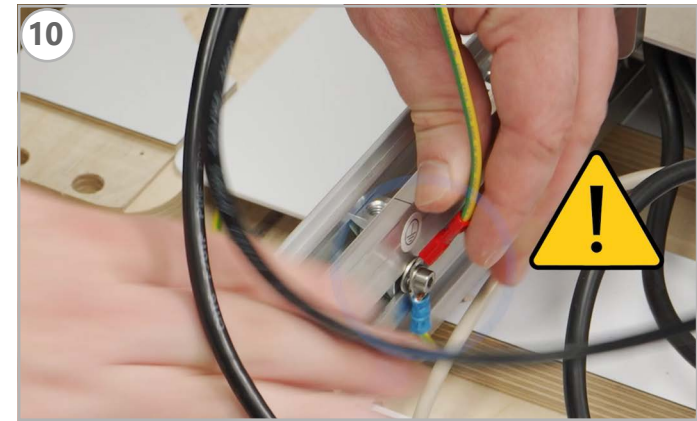
→ Connect the protective conductor cable of the lifting column to the grounding point of the table top.


→ Connect the control cable for the lifting column.

→ Connect power cable for lifting column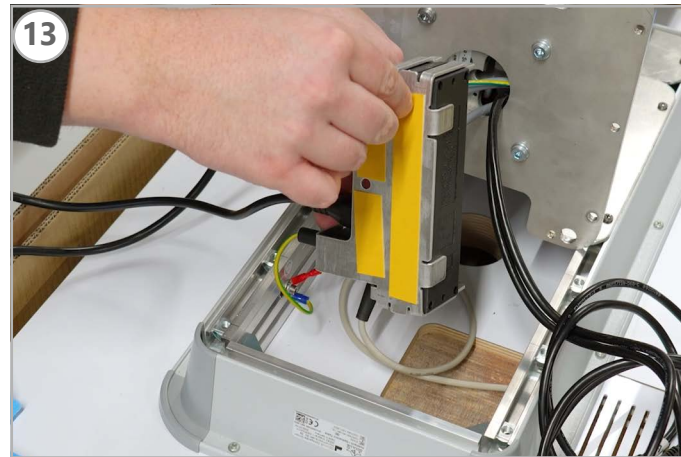


➔ Remove protective film from the adhesive tape on the power strip.

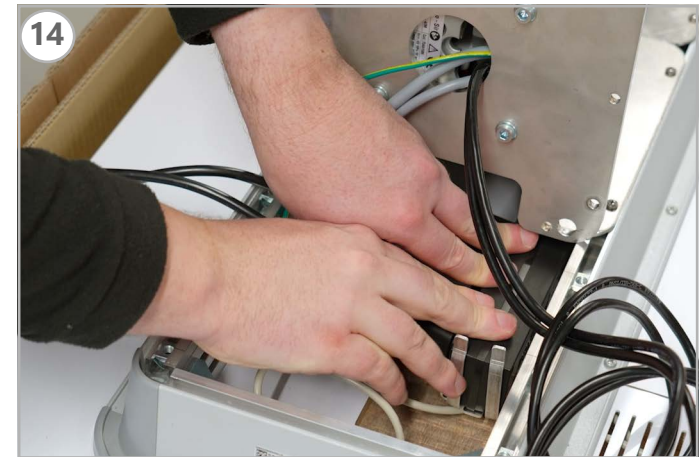
➔ Fasten power strip inside the frame.
➔ Stow all cables in the frame without kinking or crushing them.


➔ Place the lifting column upright.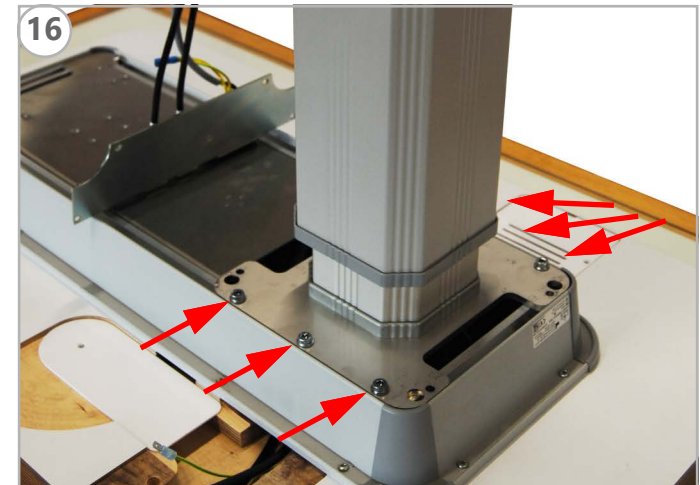

➔ Screw on tightly the lifting column onto the table top using 6 x M6x16 Allen screws, use the second tool to apply more force.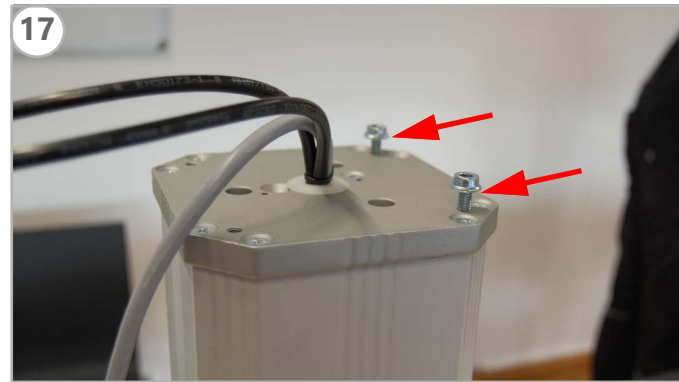


→ To attach the table base, first loosely screw 2 x M6x16 Allen screws into the lifting column.
Pay attention to the correct direction! [...]

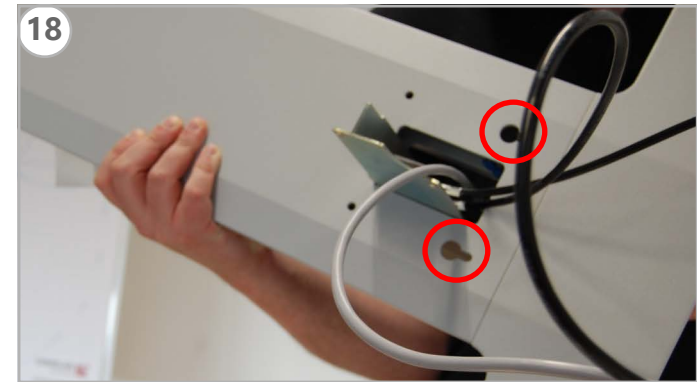
[...] Screw in the screws on the same side of the lifting column as the long holes in the table base so that the table base can be placed on the lifting column.


→ Place the table base vertically on the table top so that the long holes and the opening for the plug plate are at the top.
→ Carefully insert the plug plate through the opening in the table base so as not to cause scratches.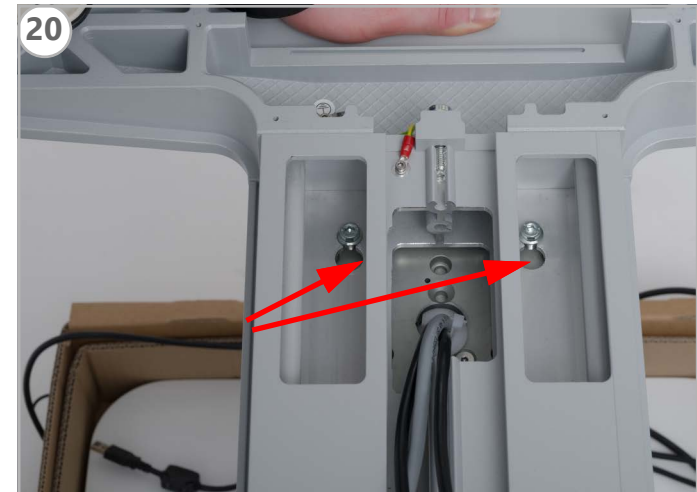

→ Place the elongated holes in the table base over the loosely attached screws of the lifting column so that the table base lies upside down on the lifting column.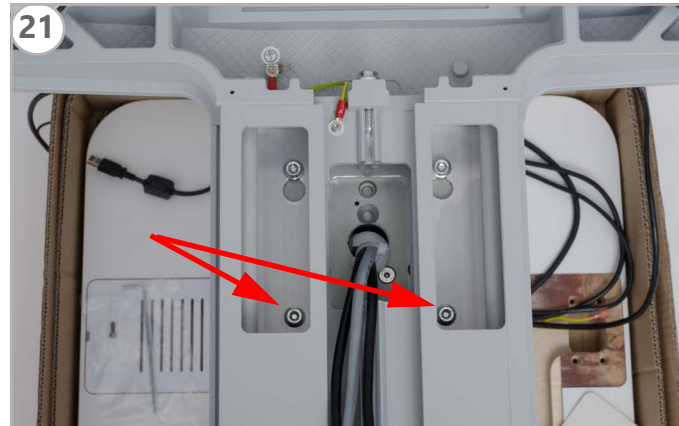

21 → Tighten the screws well and screw in 2 more M6x16 Allen screws.

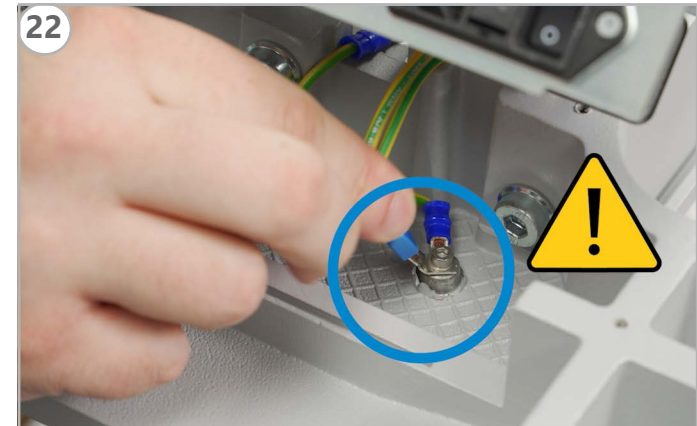22 → Attach the protective conductor cable of the connector plate to the grounding point of the table base.

23 → Position the connector strip exactly in the designated groove and lay the cable.


24 → Tighten the connector strip with 4x M3x6 countersunk Philips-head screws.


→ Store cables in the cable duct without bending or crushing them.

→ Insert the cable duct cover into the guiding.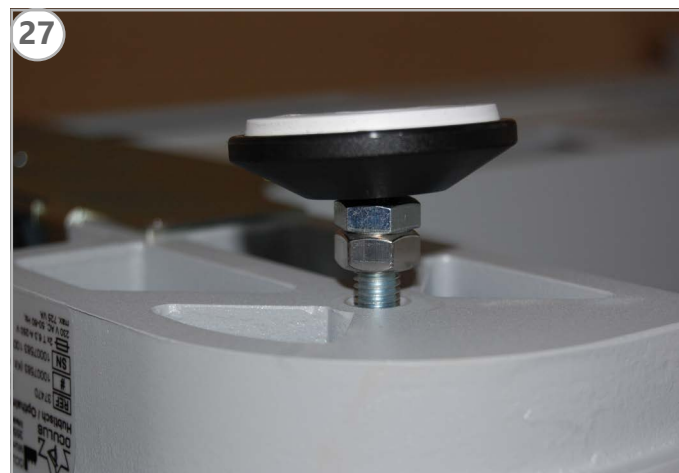

→ Screw the standard device foot (see fig.) or rollers (optional) into the table base.

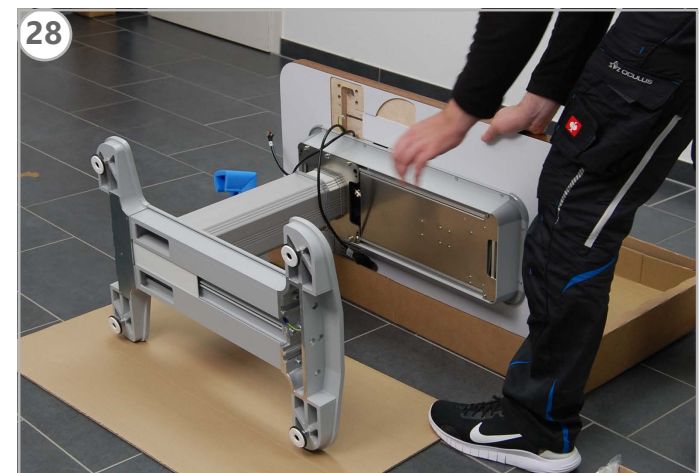→ To set up the table, first lay it on its side.

→ And then set up the table.

→ Connect the power cable and plug it into the power supply.
→ Test that the ophthalmic table is working properly.

**Need
assistance?**
Scan & See
our assembly
tutorial

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OCULUS Optikgeräte GmbH