
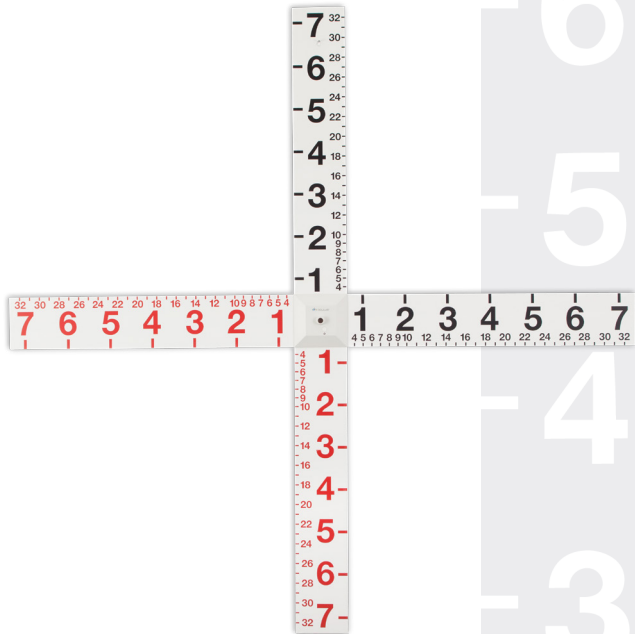
# OCULUS

## Maddox-Kreuz

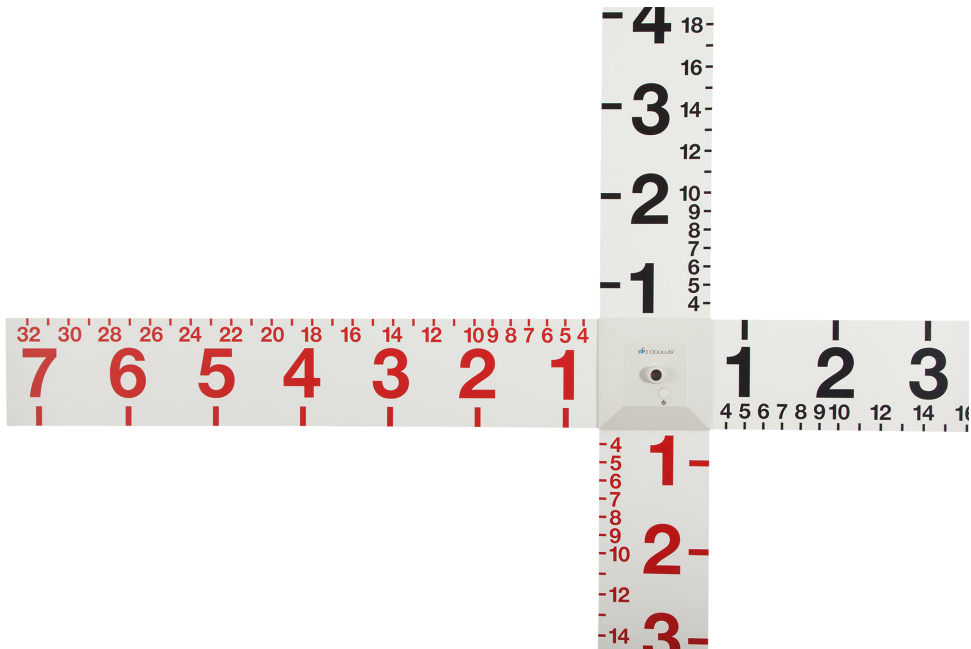Gebrauchsanweisung  
Instruction Manual  
Manual de instrucciones  
Notice d'utilisation  
Istruzioni per l'uso

DE

Konformität | Conformity | Conformidad | Conformité | Conformità

EN

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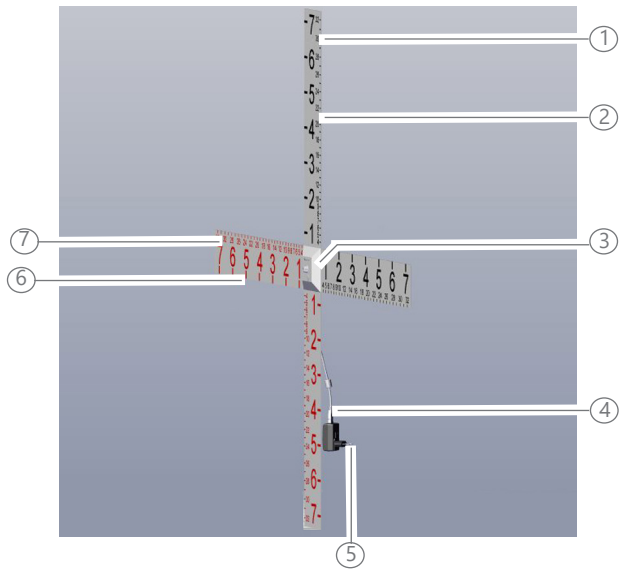
EN

## Overview

- 1 Mounting hole
- 2 Scale arm
- 3 Light with On/Off switch
- 4 USB port
- 5 USB plug-in power adaptor
- 6 Angular degrees at a testing distance of 5 m
- 7 Angular degrees at a testing distance of 1 m

## Proper Use

The Maddox cross is a tangent scale for determining the strabismus angle and testing the correspondence.

## Scope of Delivery

- Scale set
- 4 Screws
- Lighting
- USB plug-in power adaptor

All rights to change the scope of delivery in the interest of ongoing technical development reserved.

## Safety Instructions

- Connect the USB connector after assembly only.
- Connect the USB connector to the power supply with the provided power adaptor. Never plug the USB connector directly into a mains power socket.
- Before cleaning the unit, disconnect the USB connector from the power supply.
- The company OCULUS Optikgeräte GmbH accepts no liability for the further use of the data determined with the Maddox cross in any form whatsoever.

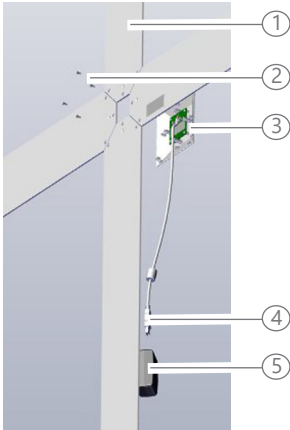
## Assembly

For the assembly, you will need

- Medium-sized Phillips-type screwdriver
- Soft underlay
- Hook or screw for wall-mounting

Mounting height: The fixation light should be at the patient's eye level  
Recommended height for nail or screw: Approx. 1.75 m

- Connect the USB connector after assembly only.
- Mount the Maddox cross in front of a non-distractive background, for example to a white wall.

- 1 Scale arm with mounting hole, blank side
- 2 Four screws
- 3 Light with On/Off switch
- 4 USB connector
- 5 USB plug-in power adaptor

- Put the components down onto the soft underlay
- Turn the light (3) upside down.
- Place the light in front of you so that the markings are correctly legible when the unit is hung up.
- First mount the scale arm with the mounting hole (1), so that the arm points upwards.
- Slide the three holes on the scale arm onto the fixing sleeves at the light. The markings of the scale arm point downwards towards the underlay.
- Fasten the scale arm into place with a screw (2).
- Proceed in the same way to fasten the other three scale arms, one after the other. When doing so, make sure that the markings are in the correct direction.
- Hang up the Maddox cross with a hook or a screw.

The fixation light must be located at eye level of the patient sitting in front of it.

- Connect the USB connector to the power supply with the supplied USB plug-in power adaptor.  
Never plug the USB connector directly into a mains power socket.

## Operation

You can conduct the following examinations, for example, with the Maddox cross:

- Determine the strabismus angle with the help of subjective localisation
- Determine the strabismus angle with light reflection
- Examine cyclo deviation using the confusion principle
- Check the correspondence

- Switch on the light at the On/Off switch.
- Conduct the desired examination.
- After completing the examination, switch off the light.

## Cleaning and Care

Switch off the light at the On/Off switch and disconnect the USB connector from the power supply.

- Clean the Maddox cross with a soft, lint-free cloth, when necessary.
- If it is very dirty, we recommend that you use a little washing up liquid on a damp cloth.
- Never use alcohol, ether or similar solvents for cleaning purposes.

## Disposal

In accordance with Directive 2012/19/EU of the European Parliament and of the Council dated 27 January 2003, and also the Law of the Federal Republic of Germany on the Commercialization, Recall and Environmentally Compatible Disposal of Electrical and Electronic Equipment, old electrical and electronic equipment must be sent out for recycling and may not be disposed in household trash.

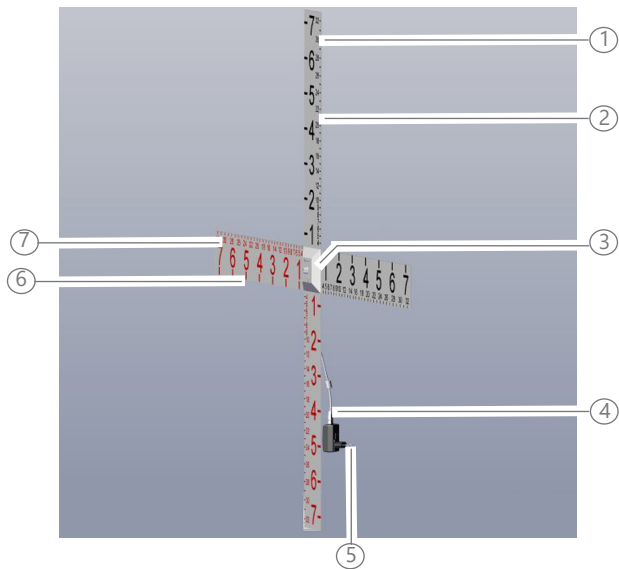
- Dispose of the Maddox cross in the proper manner.

## Guarantee

Our guarantee complies with legal requirements from the date of delivery. This guarantee covers all malfunctions and defects caused by material defects and poor workmanship. If you have cause to lodge justified complaints during the guarantee period, these will be rectified free of charge. The guarantee does not cover normal wear and tear, negligent use, or the use of spare parts other than genuine OCULUS spare parts.

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