OCULUS

Spiegel-Exophthalmometer

nach Hertel

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Instruction Manual

Manual de Instrucciones

Mode d'emploi

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Konformität | Conformity | Conformidad | Conformité | Conformità

de

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en

Contents

Technical data	17
Scope of Delivery	17
Overview.....	18
Product Description	19
Intended Use.....	19
Operation.....	20
Cleaning and Care.....	21
Warranty.....	21

en

Technical data

Dimensions (H x W x D)	26 x 260 x 78 mm
Weight	111 g
Scale on track	75 - 135 mm
Inner scale mirror	0 - 35 mm


Scope of Delivery

Hertel Mirror Exophthalmometer	52400
Instruction Manual	GA/52400/IN

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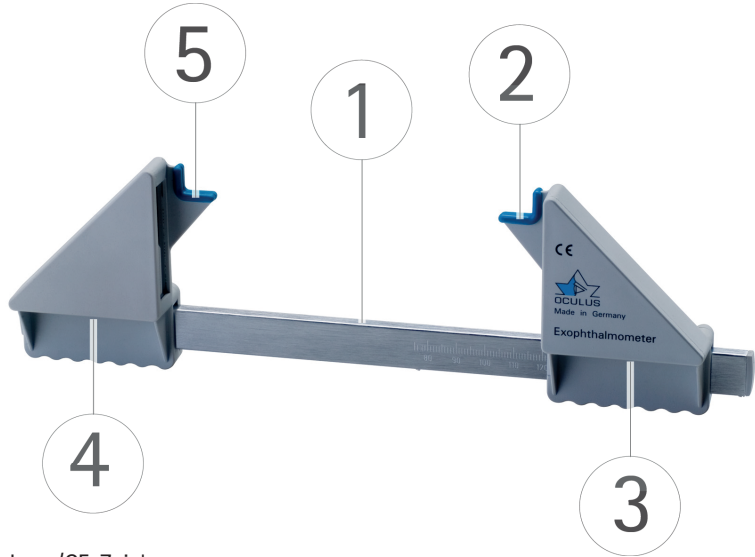
Product Description

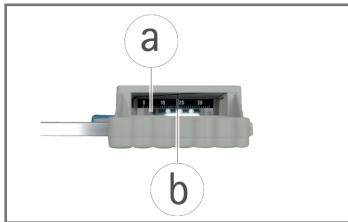- 1 Bar with scale
- 2 blue arced support (left orbital wall)
- 3 Triangular mirror holder for left eye, with inner scale
- 4 Red fixation line in the mirror holder
- 5 blue arced support (right orbital wall)

- a Mirror with scale
- b Red fixation line in the mirror housing

Intended Use

The Hertel mirror exophthalmometer is used for precise measurement of the degree of forward displacement of the eye in the sagittal axis (proptosis). The distance between the lateral orbital rim and the apex of the cornea serves as the dimension of the measurement.

Operation

- Sit down opposite the patient.
- Using both hands, position the blue supports of the mirror exophthalmometer (2, 5) at the temporal lateral orbital walls. Make sure that the exophthalmometer is initially firmly propped against the right orbital rim (whereby the edge of the bone should be felt against the lowest point of the blue support).
- With your right hand, adjust the moveable part on the slide rail so that the left orbital rim is also positioned at the lowest point of the blue support.
- Take a reading of the orbital rims at the top on the scale on the sliding rail (1).
- Note down this value for future measurements.
- Ask the patient to open his eyes wide and to look straight ahead.
- Start by measuring the exophthalmos of patient's right eye.
- With one eye, look into the left mirror housing at the mm graduation on the inner scale
- Move your head from side to side until the red fixation line is located at the 18 mm mark.
- Take a reading of the mm mark at which the patient's corneal apex is located.
- Now conduct the measurement of the exophthalmos of the left eye.

Cleaning and Care

- Clean the Mirror Exophthalmometer with a soft, lint-free cloth, when necessary.
- If it is very dirty, we recommend: Use a little washing-up liquid on a damp cloth.
- Never use alcohol, ether or similar solvents for cleaning purposes.
- Disinfect the Mirror Exophthalmometer before every use.

For disinfection, we recommend the use of:

Pursept®-A Xpress disinfectant wipes from the company Merz, or Mikrozyd® sensitive wipes, from the company Schülke & Mayr. GmbH.

Warranty

Our warranty conforms to prevailing legal regulations and is effective from the date of delivery. It covers all types of malfunctioning, as well as faults related to materials or manufacture. All legitimate claims within the warranty period will be met free of charge. This warranty shall not cover cases of normal wear and tear, improper use or use of non-OCULUS replacement parts. Please present your receipt and the delivery note when making warranty claims.n.

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OCULUS Optikgeräte GmbH