




# Risk Assessment for COVID-19 Infection using Non-Contact Tonometer

NIDEK CO., LTD.

May 29, 2020

# Objective

- To state our perspective on the risk of COVID-19 using Non-Contact Tonometry (NCT).
- To summarize and share the outcome of related studies.

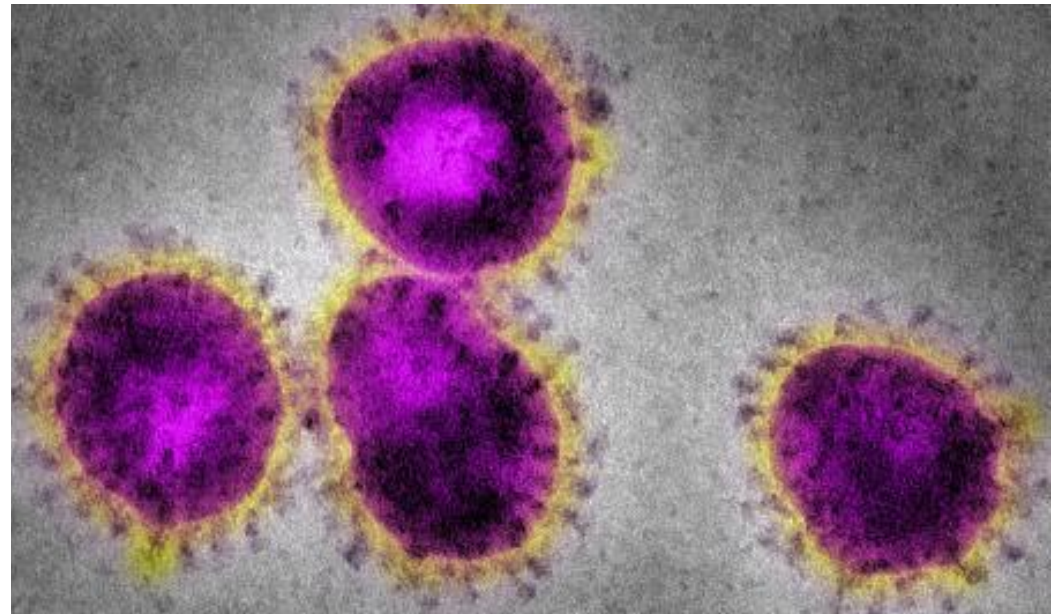

# Investigation Sources

## ■ Literature Research

Data Sources:

- PubMed
- Cochrane Database of Systematic Reviews
- Meta-analysis and reviews of health technology assessment (HTA) institutes and networks

Search Period: past 5 years

## ■ Official announcements by Public Organizations

# Study1 Relevance between COVID-19 and conjunctivitis

- 3.3% (1 out of 30) patients with COVID-19 had conjunctivitis [1]
- 0.8% (9 out of 1099) patients with COVID-19 had conjunctivitis [2]
- 2.8% (2 out of 72) patients with COVID-19 had conjunctivitis [3]
- 1.5% (1 out of 63) patients with COVID-19 had conjunctivitis [4]

## Summary:

0.8 to 3.3% patients with COVID-19 had conjunctivitis

# Study2 Presence of virus in conjunctiva region

- Positive result in conjunctival sac (3 out of 30 patients: 1 with conjunctivitis and 2 without conjunctivitis). [5]
- Positive result by conjunctival swab test (2 out of 38 patients). [6]

## Summary:

- 3.6 to 16.7% patients with conjunctivitis had COVID-19
- COVID-19 was detected in conjunctiva region having no conjunctivitis

# Study3 Presence of virus in tears

- Positive result in tears (1 (with conjunctivitis) out of 30 patients).  
[7]
- No virus was detected in tears out of 17 patients with COVID-19. [8]
- Citing the above study [8] , the risk of infection through tears is low, however the study suggests that tears may cause COVID-19 transmission. [9]
- Citing the above study [8] , the risk of infection through tears is low. [10]

## Summary:

- Patients with COVID-19 in tears is 3.3% or less

# Study4 Risk of airborne transmission

- The main route of COVID-19 transmission is respiratory droplets and indirect contact, however infection via aerosol or tears has not been reported. [11]
- COVID-19 is a droplet infection, however, no infectious particles in the air have been detected. [12]

## Summary:

- Infection via aerosol or tears is not detected

# Study5 WHO announcement

- Current evidence from WHO that COVID-19 is transmitted among people mainly through respiratory droplets and contact routes. [13]
- WHO states that no airborne transmissions were reported in an analysis of 75,465 COVID-19 cases. [14]

## Summary:

- COVID-19 is transmitted through respiratory droplets and contact routes, no airborne transmission is reported



# Study6 Transmission using NCT

- NIDEK could not confirm any literature indicating transmission of infectious diseases by NCT examination
- NIDEK has been manufacturing and marketing NCT for many years and we have not received any reports that infectious disease has been transmitted from NCT.
- The main route of COVID-19 transmission is reported through respiratory droplets and indirect contact. Human respiration volume is approx. 480-600 mL ( tidal value: 8 to 10mL/kg <sup>[15]</sup> , weight: 60kg weight). On the other hand, the maximum air puff volume of NCT is 14 mL and is less than 3% of the human respiration volume.

## Summary:

- Transmission using NCT is not reported
- Air puff volume of NCT is less than 3% of breathing

# Conclusion

- 0.8 to 3.3% patients with COVID-19 had conjunctivitis
- 3.6 to 16.7% patients with conjunctivitis had COVID-19
- Patients with COVID-19 in tears is 3.3% or less
- COVID-19 is transmitted through respiratory droplets and contact routes, no airborne transmission is reported
- Transmission using NCT is not reported
- Air puff volume of NCT is less than 3% of breathing

Risk for COVID-19 Infection using NCT

= **Very unlikely to be affected**

# Instructions for using NCT

- Clean patient contacting areas frequently
- Before using a disinfectant on NCT, soak the cloth in a solution diluted with neutral detergent, and squeeze it well and wipe off the area
- Disinfectants that can be used for NCT are Ethanol, IPA (Isopropyl Alcohol), and Hypochlorous Acid Water.  
When wiping up with disinfectant, wipe gently
- Ask the patient to wear a mask
- NIDEK will provide a Breath Shield for NCT and TONOREF III

# References

1. Jianhua Xia MM, Jianping Tong MD, Mengyun Liu MM, et al. Evaluation of coronavirus in tears and conjunctival secretions of patients with SARS - CoV - 2 infection(February 26, 2020).
2. W. Guan, Z. Ni, Yu Hu, et al. Clinical Characteristics of Coronavirus Disease 2019 in China(February 28, 2020).
3. Xufang Sun, Xian Zhang, Xuhui Chen, et al. The infection evidence of SARS-COV-2 in ocular surface: a single-center cross-sectional study(February 26, 2020).
4. Yunyun Zhou,View ORCID Profile, Yuyang Zeng, et al. Ophthalmologic evidence against the interpersonal transmission of 2019 novel coronavirus through conjunctiva(February 12, 2020).
5. A-Yong Yu, Ruixue Tu, Xu Shao, et al. A comprehensive Chinese experience against SARS-CoV-2 in ophthalmology(April 7, 2020).
6. Ping Wu, Fang Duan, Chunhua Luo, et al. Characteristics of Ocular Findings of Patients With Coronavirus Disease 2019 (COVID-19) in Hubei Province, China(March 31, 2020).
7. Jianhua Xia MM, Jianping Tong MD, Mengyun Liu MM, et al. Evaluation of coronavirus in tears and conjunctival secretions of patients with SARS - CoV - 2 infection(February 26, 2020).
8. Ivan Yu Jun Seah, Danielle E. Anderson, Adrian Eng Zheng Kang, et all. Assessing Viral Shedding and Infectivity of Tears in Coronavirus Disease 2019 (COVID-19) Patients(March 24, 2020).
9. Soumen Sadhu, B.Optom, Rupesh Agrawa, et al. COVID-19: Limiting the Risks for Eye Care Professionals(April 20, 2020).
10. Pietro Emanuele Napoli, Matteo Nioi, Ernesto d'Aloja, et al. The Ocular Surface and the Coronavirus Disease 2019: Does a Dual 'Ocular Route' Exist?(April 28, 2020 ).
11. Yang Chao, Ma Qiuyan, Zheng Yuhong, et all. Transmission routes of 2019-novel coronavirus(2019-nCoV) (April 6, 2020).
12. Jakob Siedlecki, Victor Brantl, Benedikt Schworm, et al. COVID-19: Ophthalmological Aspects of the SARS-CoV 2 Global Pandemic(April 23, 2020).
13. World Health Organization (WHO). Q&A on coronaviruses(COVID-19)(17 April, 2020)
14. World Health Organization (WHO). Modes of transmission of virus causing COVID-19: implications for IPC precaution recommendations(March 27, 2020).
15. K.Suwa, Radiometer Medical ApS. Basics of artificial respiration: Ventilation volume and frequency. Tidal volume:8~10ml/kg(2020).